


## Device Preparation

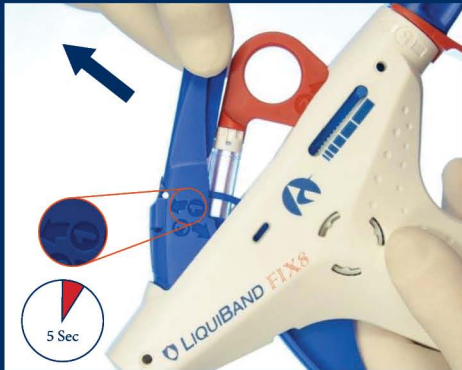

### Step 1:

Hold the device pointing down at approximately 45°. Lift the blue cover firmly to its full extent in the direction of arrow 1 to completely crush the ampoule.

Wait for 5 seconds following the ampoule crush.



### Step 2:

Hold the device with the applicator body vertical, and the tip pointing down, draw the red transfer pull component backwards very slowly. **Full transfer of tissue adhesive to the dispensing chamber should take approximately 20 seconds.** At the end of its travel the transfer pull component will disengage automatically and can be discarded.



### Step 3:

Rotate down the blue cover component in the direction of arrow 3 and push firmly until it clicks into the device body.



### Step 4:

Remove the red priming lock component and discard.



### Step 5:

Ensure the distal tip of the device is located in plastic tray dispensing well, turn the rotatory knob component in the direction shown by arrow 5. The trigger will automatically be released.



### Step 6:

Keeping the distal tip within the dispensing cavity actuate the trigger until tissue adhesive is dispensed (Approximately 2 to 3 times). The device is now primed and ready for use.